

Kit Instructions



- 1** To open collection cup, tear shrink banding along perforations, which are marked with printed arrows.



- 2** Remove vials and bag from collection cup.



- 3** Top of collection cup can be used in collection of the specimen.



- 4** **Single Sample** - Tear shrink band along perforations.



- OR** **Split Sample** - Simply twist vials in opposite direction to break tamper evident shrink banding.



Kit Instructions *cont.*



- 5** Snap vial closed completely.



- 6** Fold lid tab down to lock on vial.



- 7** Place Custody and Control form seals over vial(s).



- 8** Insert vial(s) into front pocket of Specimen bag.



- 9** Take completed copy of Custody and Control form and insert into rear pocket.



- 10** Remove release liner from bag and seal bag.

